## Flushing Medium

### Ingredientia

#### 1084 Flushing Medium

Albumini humani solutio; Aqua; CaCl<sub>2</sub>; Gentamicinum;  
Glucosum; HEPES; KCl; MgSO<sub>4</sub>; NaHCO<sub>3</sub>; NaCl;  
NaH<sub>2</sub>PO<sub>4</sub>; Phenolsulfonphthaleinum; Pyruvas; SSR

#### 1076 Flushing Medium with Heparin

Albumini humani solutio; Aqua; CaCl<sub>2</sub>; Gentamicinum;  
Glucosum; Heparinum; HEPES; KCl; MgSO<sub>4</sub>; NaCl;  
NaHCO<sub>3</sub>; NaH<sub>2</sub>PO<sub>4</sub>; Phenolsulfonphthaleinum; Pyruvas;  
SSR;

#### Product No.

1084  
1076

#### Customer Service:

E-mail: [customerservice.medicult@origio.com](mailto:customerservice.medicult@origio.com)

Tel: +45 46 79 02 02 Fax: +45 46 79 03 02

ORIGIO a/s

Knardrupvej 2, DK-2760 Måløv, Denmark

[www.origio.com](http://www.origio.com)

Tel: +45 46790200 Fax: +45 46790300

**origio** MEDICULT  
'S MEDIA

en

## Flushing Medium

---

For retrieval, holding and washing of oocytes.

This product is for IVF treatment of women, whether the cause of infertility is male or female. The product should only be used by professionals trained in IVF treatment.

### Products

1084 Flushing Medium

1076 Flushing Medium with 10 IU/ml Heparin

### Contains

Heparin 10 IU/ml (not Product No. 1084)

Human albumin solution (HAS)

Recombinant human insulin

Gentamicin sulphate 10 µg/ml

### Quality control testing

Sterility tested (Ph.Eur., USP)

Osmolality tested (Ph.Eur., USP)

pH tested (Ph.Eur., USP)

Endotoxin tested  $\leq 0.1$  EU/ml (Ph.Eur., USP)

HSA analysis (Ph.Eur, USP)

Mouse Embryo Assay (MEA) tested

**Note:** The results of each batch are stated on a Certificate of Analysis, which is available on [www.origio.com](http://www.origio.com).

### Storage instructions and stability

The products are aseptically processed and supplied sterile.

Store in original container at 2-8°C, protected from light.

Do not freeze.

Discard excess (unused) media following warming.

The product is to be used within 7 days after opening.

When stored as directed by the manufacturer the product is stable until the expiry date shown on the vial label.

### Precautions and warnings

Do not use the product if:

1. Product packaging appears damaged or if the seal is broken.
2. Expiry date has been exceeded.
3. The product becomes discoloured, cloudy, turbid, or shows any evidence of microbial contamination.

**Caution:** All blood products should be treated as potentially infectious. Source material used to manufacture this product were tested and found non-reactive for HbsAg and negative for Anti-HIV-1/-2, HIV-1, HBV, and HCV. Furthermore source material have been tested for parvovirus B19 and found to be non-elevated. No known test methods can offer assurances that products derived from human blood will not transmit infectious agents.

**Caution:** Oocytes are extremely sensitive to pH changes and transient cooling in vitro. Modest fluctuations in temperature can cause disruption of the meiotic spindle with possible chromosome dispersal.

**Caution:** This product contains Gentamicin and should not be used on patients that have known allergy to Gentamicin or similar antibiotics.

**Note:** The cap of the bottle should be kept tightly closed during the pre-warming.




**Note:** Please note the need for traceability of this product. In addition national legal requirements in your country may exist in this field.

**Note:** Devices used in combination with this device should be intended for the particular purpose.

**Note:** Dispose of the device in accordance with local regulations for disposal of medical devices.

### Instructions for use

1. Pre-warm Flushing Medium to 37°C.
2. Keep Flushing Medium at 37°C in a heating block throughout the entire process of oocyte recovery and transport to the embryology laboratory.
3. Syringes used for flushing the follicle are filled with the Flushing Medium. Aspirate the follicular fluid and flush the follicle with a volume of medium equivalent to the amount of follicular fluid collected. Take care not to expand the follicle beyond its original size.
4. Wash oocytes with Flushing Medium or pre-equilibrated culture medium, taking care to remove any blood clots and granulosa cells.
5. Transfer oocytes to pre-equilibrated culture medium and store at 37°C in a 5-6% CO<sub>2</sub> environment.

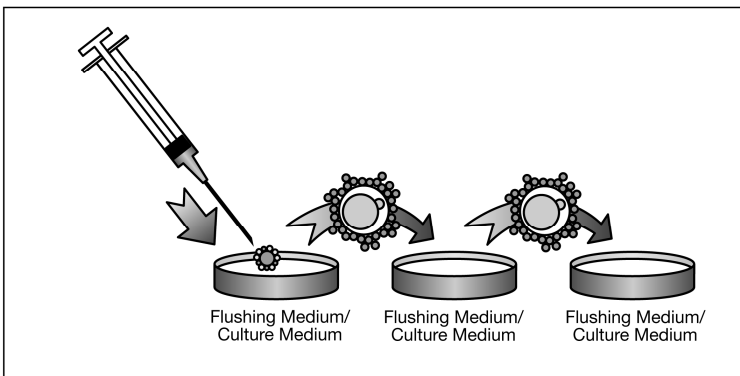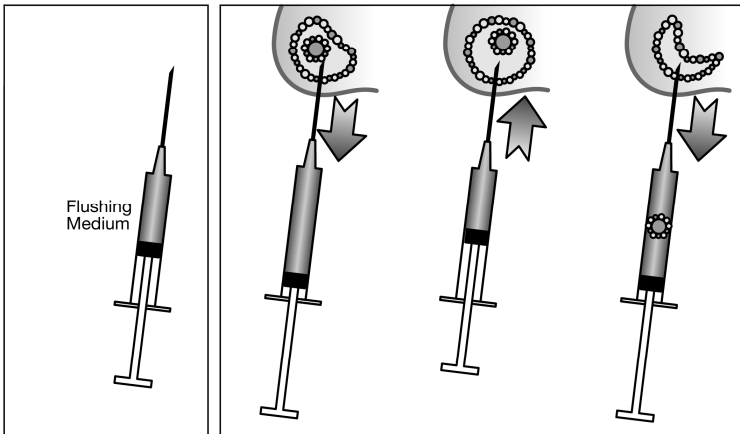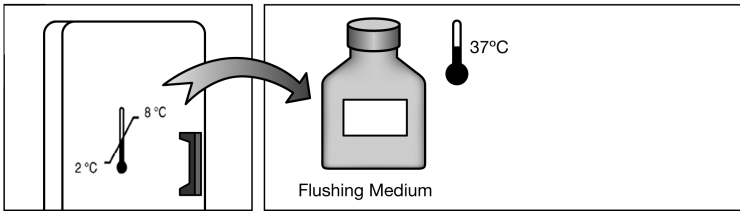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