LOGO

Shangxian Minimal Invasive Inc

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**Advise on action to be taken by the user:**

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Shangxian Minimal Invasive Inc.  
1st Floor, Block B2-2, China  
Medicine Innovation Park,  
Mulan road, Hi-tech development zone,  
117004 Benxi, Liaoning,  
China

## URGENT FIELD SAFETY NOTICE FSCA20240110/ FSCA20240110

10<sup>th</sup> January 2024

Dear Sirs

Shangxian Minimal Invasive Inc. decided to withdraw certain products from the Polish market:

**Disposable hemostatic clipper, 360° rotatable  
Disposable Hemostasis Clips and Delivery System**

Model: SX-H-2300-2 (Lot: 115304061)

Model: SX-H-2300-2 (Lot: 115308231)

Model: SX-H-2300-1 (Lot: 115309011)

### **Description of the problem:**

We are sending this letter to inform you that the importer of the SX series hemoclips, i.e. Meditech X-Ray Spółka z o.o., received a notification in which the user of the SX hemoclips reported problems when using the above-mentioned hemoclips, i.e.:

hemoclips do not close properly. They slip off the tissue.

The passage of the clips through the endoscope channel is difficult in the distal part.

The clips do not release from the distal tip. Attempting to release it may result in increased bleeding.

### **Potential risks associated with continued use:**

Increased bleeding or tissue damage that threatens the health and life of patients

Endoscope damage.

Taking into account the identified risk, we decided to withdraw all at-risk products from the market.

The recall is limited only to the above-mentioned series of products. This does not apply to other series or products.

### **Advise on action to be taken by the user:**

Advise on action to be taken by the user:

We kindly ask you to take the following actions immediately and as a matter of priority:

• Please read the full content of this Safety Notice and ensure that all users of the products in question in your organization and other interested persons are informed about the contents of the Notice

• Please identify, quarantine and return the products in question

• Please stop using the products in question

• Please confirm receipt of this notification by completing the confirmation form below and returning it to [endoskopia@meditech.pl](mailto:endoskopia@meditech.pl) using the contact details provided below:

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