

## **PILNE: UWAGA DOTYCZĄCA BEZPIECZEŃSTWA**

**Bivona® dla noworodków i dzieci:** Bivona® Aire-Cuf®, TTS™, bez mankietu, średni zakres  
**Rurki tracheostomijne Bivona® dla dorosłych:** Bivona Aire-Cuf®, TTS™, FlexTend™ bez mankietu, TTS™ FlexTend™

13 czerwca 2024 r

Szanowni Klienci Bivona®,

Firma Smiths Medical publikuje niniejszą pilną notatkę dotyczącą bezpieczeństwa, aby powiadomić Państwa o potencjalnych wadach następujących produktów do tracheostomii Bivona® dla noworodków/dzieci i dorosłych wymienionych w *Załączniku 1\_Produkt, którego dotyczy problem*. W tym piśmie szczegółowo opisano problem i kroki, które należy wykonać.

### **Wydanie:**

Firma Smiths Medical zaobserwowała, że kołnierz mocujący w określonych seriach produktów tracheostomijnych Bivona® dla noworodków/dzieci i dorosłych może pęknąć z powodu wady produkcyjnej. Na poniższym zdjęciu przedstawiono przykład rozerwanego kołnierza.



### **Potencjalne ryzyko:**

Jeżeli kołnierz przedmiotu jest rozdarty lub złamany, rurka tracheostomijna może nie pozostać na swoim miejscu w tchawicy. Może to prowadzić do przemieszczenia tracheostomii lub dekaniulacji. Każde z tych zdarzeń może spowodować brak właściwej wentylacji lub ochrony dróg oddechowych. Jedno i drugie może przyczynić się do katastrofalnego zdarzenia niepożądanego. Do chwili obecnej firma Smiths Medical otrzymała trzydzieści pięć (35) raportów o poważnych obrażeniach i jednym (1) zgonie związanym z tym problemem.

### **Produkt, którego dotyczy problem**

Proszę sprawdzić numery pozycji i partii, których dotyczy problem, w *Załączniku 1\_Lista produktów, których dotyczy problem*, a także zakres dat dystrybucji produktów. Żaden produkt otrzymany po 13 czerwca 2024 r. nie jest uważany za wadliwy.

### **Działania medyczne Smiths:**

Smiths Medical wysłała to powiadomienie do wszystkich klientów Bivona®, którzy otrzymali produkty od Smiths Medical wymienione w *Załączniku 1\_Produkt, którego dotyczy problem*. Smiths Medical zapewni produkt(y) zastępczy lub wystawi notę korygującą zainteresowanym klientom po otrzymaniu wypełnionego formularza odpowiedzi potwierdzającego zniszczenie produktu.

**Działania wymagane przez klienta :**

Podczas korzystania z urządzenia należy ze szczególną uwagą przestrzegać wszystkich instrukcji, łącznie z ostrzeżeniami zawartymi w Dokumentacji Instrukcji obsługi. Wykonaj następujące czynności wymienione poniżej

- 1) Sprawdź wszystkie lokalizacje magazynowe w swojej instytucji pod kątem numerów katalogowych i numerów partii, których to dotyczy, wymienionych w powiadomieniu i zaprzestań używania. Wyrzuć wszystkie produkty, których to dotyczy, zgodnie z procedurą utylizacji obowiązującą w Twojej instytucji. Jeżeli w Państwa zakładzie nie jest możliwa natychmiastowa utylizacja, produkt należy poddać kwarantannie do czasu utylizacji.
- 2) Udostępnij to powiadomienie wszystkim potencjalnym użytkownikom urządzenia, aby upewnić się, że znają to powiadomienie i proponowane rozwiązania zaradcze. Jeśli urządzenia są używane w innym miejscu, należy upewnić się, że niniejsza komunikacja została tam dostarczona.
- 3) Wypełnij i odeślij załączony Formularz odpowiedzi klienta na [adres EMEA-FSN@icumed.com](mailto:adres EMEA-FSN@icumed.com) w ciągu 10 dni od otrzymania, aby potwierdzić zrozumienie niniejszego powiadomienia.
- 4) **DYSTRYBUTORZY:** Jeżeli rozdystrybuowałeś wśród swoich klientów produkty, których potencjalnie dotyczy problem, prosimy o natychmiastowe przekazanie im niniejszego powiadomienia i poproszenie o wypełnienie formularza odpowiedzi i odesłanie go **TOBIE** . Następnie **DYSTRYBUTOR** musi wypełnić POJEDYNCZY formularz z wymaganymi danymi i odesłać go na [adres EMEA-FSN@icumed.com](mailto:adres EMEA-FSN@icumed.com)

W przypadku dalszych pytań prosimy o kontakt ze Smiths Medical, korzystając z następujących informacji:

Kontakt medyczny Smiths	Informacje kontaktowe	Obszary wsparcia
Globalne zarządzanie reklamacjami	<a href="mailto:globalcomplaints@icumed.com">globalcomplaints@icumed.com</a>	Aby zgłosić zdarzenia niepożądane lub reklamacje dotyczące produktu
Uwaga dotycząca bezpieczeństwa	<a href="mailto:EMEA-FSN@icumed.com">EMEA-FSN@icumed.com</a> lub skontaktuj się z przedstawicielem handlowym	Pytania dotyczące niniejszej notatki dotyczącej bezpieczeństwa

Organ regulacyjny w Twoim kraju został powiadomiony o tym działaniu

Firma Smiths Medical dba o bezpieczeństwo pacjentów i koncentruje się na zapewnianiu wyjątkowej niezawodności produktów i najwyższego poziomu zadowolenia klientów. Dziękujemy za szybkie wsparcie w tej ważnej sprawie. Doceniamy twoją współpracę.

Z poważaniem,



Andy Matheina  
Wiceprezes ds. Jakości

**Obudowy:**

- Formularz odpowiedzi klienta (patrz poniżej)
- Lista produktów, których dotyczy problem (załącznik 1)

## PILNE: UWAGA DOTYCZĄCA BEZPIECZEŃSTWA – FORMULARZ ODPOWIEDZI

**Bivona® dla noworodków i dzieci:** Bivona® Aire-Cuf®, TTS™, bez mankietu, średni zakres  
**Rurki tracheostomijne Bivona® dla dorosłych:** Bivona Aire-Cuf®, TTS™, FlexTend™ bez mankietu, TTS™ FlexTend™

13 czerwca 2024 r

**Sprawdź swoje zapasy i uzupełnij poniższe informacje, nawet jeśli nie posiadasz produktu, którego dotyczy problem.**  
Niewypełnienie wszystkich sekcji tej strony może skutkować nieprawidłowym, opóźnionym lub odmową wystawienia noty korygującej.

Wypełniony formularz prosimy odesłać na adres [EMEA-FSN@icumed.com](mailto:EMEA-FSN@icumed.com). Jeśli masz pytania dotyczące tego formularza, skontaktuj się z [EMEA-FSN@icumed.com](mailto:EMEA-FSN@icumed.com) lub lokalnym przedstawicielem handlowym.

Nazwa szpitala/obiektu	
Adres szpitala/placówki	
Numer telefonu	
Imię i nazwisko oraz stanowisko osoby wypełniającej ten formularz	
Podpis osoby wypełniającej niniejszy formularz	
Data	
W przypadku zakupu za pośrednictwem dystrybutora proszę podać tutaj nazwę/lokalizację dystrybutora w celu zapewnienia identyfikowalności	

Proszę wybrać jeden:

- NIE** mam produktów, których dotyczy problem (wypełnij i odeślij formularz na powyższy adres e-mail)
- TAK**, mam produkty, których dotyczy problem, powiadomiłem użytkowników w mojej placówce, postępowalem zgodnie z otrzymanymi instrukcjami i zniszczyłem wszystkie produkty, których dotyczy problem (patrz tabela poniżej)

Jeżeli posiadasz produkt, którego dotyczy ten problem, wypełnij poniższą tabelę:

TABELA 1

Numer partii	Ilość w magazynie (szt.)	Ilość zniszczonych (każda sztuka)	Data zniszczenia	PO, notatka debetowa lub faktura

Jeśli rozprowadzałeś produkt dalej, wypełnij poniższą tabelę zebranymi informacjami otrzymanymi od klientów i przekaż ICU Medical ogólne informacje.

TABELA 2

Numer partii	Ilość zniszczona lokalnie (szt.)	Data zniszczenia

Zdarzenia niepożądane i skargi związane ze stosowaniem tego produktu należy zgłaszać i przysyłać pocztą elektroniczną do Globalnego Działu Zarządzania Skargami firmy Smiths Medical na adres [globalcomplaints@icumed.com](mailto:globalcomplaints@icumed.com).

<b>Affected items below were distributed in the Poland between July 2019 and Apr 2024</b>		
<b>SKU/List/Model Number</b>	<b>Product Description</b>	<b>Lot/Serial Number</b>
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	3782488
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	3790083
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	3790086
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	3790092
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	3800546
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	3831843
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	3870024
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	3890396
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	3901138
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	3952162
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	3952176
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	3984878
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4002336
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4002350
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4039104
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4083955
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4111664
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4122545
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4145805
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4148058
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4151118
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4151119
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4168074
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4205245
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4231601
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4231603
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4242052
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4244548
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4248369
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4257848
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4267574
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4267576
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4267577
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4270315
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4279289
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4283178
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4291368
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4291369
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4301891
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4331666

670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4335526
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4362629
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4371685
670170	TTS TRACHEOSTOMY TUBE 7.0MM TIGHT TO SHAFT CUFF	4371688
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	3790011
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	3794176
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	3801426
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	3828139
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	3849161
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	3852712
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	3852713
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	3869836
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	3890270
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	3890282
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	3890292
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	3890301
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	3894724
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	3894730
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	3914501
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	3917889
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	3935873
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	3939393
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	3946138
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	3946142
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	3946145
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	3952258
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	3952260
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	3984935
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4002252
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4002275
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4026724
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4026728
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4033783
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4033784
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4033785
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4035851
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4039106
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4071071
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4086071
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4089228
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4101022
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4131046
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4139052
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4142498
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4145763

670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4151125
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4160012
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4160013
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4160015
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4162026
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4237417
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4240189
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4242055
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4257856
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4265148
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4265150
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4267578
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4276389
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4285198
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4291916
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4297024
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4297029
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4321558
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4321559
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4335535
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4358884
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4358886
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4371684
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4394609
670180	TTS TRACHEOSTOMY TUBE 8.0MM TIGHT TO SHAFT CUFF 1/EA	4396612
750170	AIRE-CUF TRACHY TUBE 7.0MM MID-RANGE	3790111
750170	AIRE-CUF TRACHY TUBE 7.0MM MID-RANGE	4033787
750170	AIRE-CUF TRACHY TUBE 7.0MM MID-RANGE	4134121
750170	AIRE-CUF TRACHY TUBE 7.0MM MID-RANGE	4142533
750170	AIRE-CUF TRACHY TUBE 7.0MM MID-RANGE	4145790
750180	AIRE-CUF TRACHY TUBE 8.0MM MID-RANGE	3778495
750180	AIRE-CUF TRACHY TUBE 8.0MM MID-RANGE	3801535
750180	AIRE-CUF TRACHY TUBE 8.0MM MID-RANGE	3801537
750180	AIRE-CUF TRACHY TUBE 8.0MM MID-RANGE	4107661
750180	AIRE-CUF TRACHY TUBE 8.0MM MID-RANGE	4110491
750180	AIRE-CUF TRACHY TUBE 8.0MM MID-RANGE	4125917
750180	AIRE-CUF TRACHY TUBE 8.0MM MID-RANGE	4131094
750180	AIRE-CUF TRACHY TUBE 8.0MM MID-RANGE	4139048
750180	AIRE-CUF TRACHY TUBE 8.0MM MID-RANGE	4291924
60N035	NEONATAL TRACH TUBE	4136144
60N035	NEONATAL TRACH TUBE	4348781
60N040	NEONATAL TRACH TUBE	3952346
60N040	NEONATAL TRACH TUBE	3962603
60N040	NEONATAL TRACH TUBE	3982698
60N040	NEONATAL TRACH TUBE	4015574

60N040	NEONATAL TRACH TUBE	4089296
60N040	NEONATAL TRACH TUBE	4142419
60N040	NEONATAL TRACH TUBE	4156652
60N040	NEONATAL TRACH TUBE	4209580
60N040	NEONATAL TRACH TUBE	4221355
60N040	NEONATAL TRACH TUBE	4233978
60N040	NEONATAL TRACH TUBE	4286814
60N040	NEONATAL TRACH TUBE	4362872
60NFP40	NEONATAL TRACHEOSTOMY TUBE 4.0FLEXTEND PLUS	3982646
60P025	PAED. TRACHEOSTOMY TUBE 2.5MM UNCUFFED	4345041
60P030	PAED. TRACHEOSTOMY TUBE 3.0MM UNCUFFED	4024940
60P030	PAED. TRACHEOSTOMY TUBE 3.0MM UNCUFFED	4028385
60P030	PAED. TRACHEOSTOMY TUBE 3.0MM UNCUFFED	4335056
60P035	PAED. TRACHEOSTOMY TUBE 3.5MM UNCUFFED	4085993
60P035	PAED. TRACHEOSTOMY TUBE 3.5MM UNCUFFED	4276479
60P035	PAED. TRACHEOSTOMY TUBE 3.5MM UNCUFFED	4283252
60P035	PAED. TRACHEOSTOMY TUBE 3.5MM UNCUFFED	4292179
60P035	PAED. TRACHEOSTOMY TUBE 3.5MM UNCUFFED	4331633
60P035	PAED. TRACHEOSTOMY TUBE 3.5MM UNCUFFED	4335059
60P035	PAED. TRACHEOSTOMY TUBE 3.5MM UNCUFFED	4369612
60P035	PAED. TRACHEOSTOMY TUBE 3.5MM UNCUFFED	4380056
60P040	PAED. TRACHEOSTOMY TUBE 4.0MM UNCUFFED	3868273
60P040	PAED. TRACHEOSTOMY TUBE 4.0MM UNCUFFED	3874304
60P040	PAED. TRACHEOSTOMY TUBE 4.0MM UNCUFFED	3894882
60P040	PAED. TRACHEOSTOMY TUBE 4.0MM UNCUFFED	3962569
60P040	PAED. TRACHEOSTOMY TUBE 4.0MM UNCUFFED	3962572
60P040	PAED. TRACHEOSTOMY TUBE 4.0MM UNCUFFED	4035762
60P040	PAED. TRACHEOSTOMY TUBE 4.0MM UNCUFFED	4082681
60P040	PAED. TRACHEOSTOMY TUBE 4.0MM UNCUFFED	4086017
60P040	PAED. TRACHEOSTOMY TUBE 4.0MM UNCUFFED	4107611
60P040	PAED. TRACHEOSTOMY TUBE 4.0MM UNCUFFED	4110467
60P040	PAED. TRACHEOSTOMY TUBE 4.0MM UNCUFFED	4122444
60P040	PAED. TRACHEOSTOMY TUBE 4.0MM UNCUFFED	4128026
60P040	PAED. TRACHEOSTOMY TUBE 4.0MM UNCUFFED	4148005
60P040	PAED. TRACHEOSTOMY TUBE 4.0MM UNCUFFED	4217451
60P040	PAED. TRACHEOSTOMY TUBE 4.0MM UNCUFFED	4233979
60P040	PAED. TRACHEOSTOMY TUBE 4.0MM UNCUFFED	4286817
60P040	PAED. TRACHEOSTOMY TUBE 4.0MM UNCUFFED	4289497
60P040	PAED. TRACHEOSTOMY TUBE 4.0MM UNCUFFED	4335063
60P040	PAED. TRACHEOSTOMY TUBE 4.0MM UNCUFFED	4348797
60P040	PAED. TRACHEOSTOMY TUBE 4.0MM UNCUFFED	4369616
60P040	PAED. TRACHEOSTOMY TUBE 4.0MM UNCUFFED	4374474
60P040	PAED. TRACHEOSTOMY TUBE 4.0MM UNCUFFED	4374475
60P040	PAED. TRACHEOSTOMY TUBE 4.0MM UNCUFFED	4381034
60P040	PAED. TRACHEOSTOMY TUBE 4.0MM UNCUFFED	4382707

60P040	PAED. TRACHEOSTOMY TUBE 4.0MM UNCUFFED	4389926
60P045	PAED. TRACHEOSTOMY TUBE 4.5MM UNCUFFED	3858169
60P045	PAED. TRACHEOSTOMY TUBE 4.5MM UNCUFFED	3868290
60P045	PAED. TRACHEOSTOMY TUBE 4.5MM UNCUFFED	3904658
60P045	PAED. TRACHEOSTOMY TUBE 4.5MM UNCUFFED	3911429
60P045	PAED. TRACHEOSTOMY TUBE 4.5MM UNCUFFED	4195807
60P045	PAED. TRACHEOSTOMY TUBE 4.5MM UNCUFFED	4206839
60P045	PAED. TRACHEOSTOMY TUBE 4.5MM UNCUFFED	4218719
60P045	PAED. TRACHEOSTOMY TUBE 4.5MM UNCUFFED	4289498
60P045	PAED. TRACHEOSTOMY TUBE 4.5MM UNCUFFED	4335067
60P045	PAED. TRACHEOSTOMY TUBE 4.5MM UNCUFFED	4352800
60P045	PAED. TRACHEOSTOMY TUBE 4.5MM UNCUFFED	4374478
60P045	PAED. TRACHEOSTOMY TUBE 4.5MM UNCUFFED	4381036
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	3946272
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	3982707
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	3982708
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	3988161
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	3991205
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	3997823
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	4005680
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	4005681
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	4015583
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	4015587
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	4085998
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	4092634
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	4107583
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	4125824
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	4204825
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	4209586
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	4218721
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	4279348
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	4286819
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	4289496
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	4292182
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	4335070
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	4335072
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	4335073
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	4352804
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	4369620
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	4369621
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	4369622
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	4374479
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	4374480
60P050	PAED. TRACHEOSTOMY TUBE 5.0MM UNCUFFED	4388210
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	3853488



60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	3897864
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	3908004
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	3908005
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	3939631
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	3952372
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	3982709
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	3988163
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	3991206
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	3991207
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	4005683
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	4015591
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	4053708
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	4071976
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	4077092
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	4107591
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	4185434
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	4201397
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	4254282
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	4254283
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	4273443
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	4286820
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	4292183
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	4327941
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	4327942
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	4335075
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	4335076
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	4335077
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	4351252
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	4374481
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	4374483
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	4380058
60P055	PAED. TRACHEOSTOMY TUBE 5.5MM UNCUFFED	4380059
60PFP55	PAED. TRACHEOSTOMY TUBE 5.5MM FLEXTEND PLUS	3880723
60PFP55	PAED. TRACHEOSTOMY TUBE 5.5MM FLEXTEND PLUS	4334996
60PFP60	6.0MM FLEXTEND PEDIATRIC PLUS VEE TRACH TUBE	3887741
60PFP60	6.0MM FLEXTEND PEDIATRIC PLUS VEE TRACH TUBE	3942474
60PFP60	6.0MM FLEXTEND PEDIATRIC PLUS VEE TRACH TUBE	4015523
60PFP60	6.0MM FLEXTEND PEDIATRIC PLUS VEE TRACH TUBE	4026688
60PFP60	6.0MM FLEXTEND PEDIATRIC PLUS VEE TRACH TUBE	4056241
60PFP60	6.0MM FLEXTEND PEDIATRIC PLUS VEE TRACH TUBE	4113112
60PFP60	6.0MM FLEXTEND PEDIATRIC PLUS VEE TRACH TUBE	4153591
60PFP60	6.0MM FLEXTEND PEDIATRIC PLUS VEE TRACH TUBE	4179355
60PFP60	6.0MM FLEXTEND PEDIATRIC PLUS VEE TRACH TUBE	4201374
60PFP55	5.5MM FLEXTEND PEDIATRIC PLUS STRIAGHT TRACH TUBE	4335002
60PFS35	PAED. TRACHEOSTOMY TUBE 3.5MM STANDARD FLEXTEND PLUS	4360407

60PFS35	PAED. TRACHEOSTOMY TUBE 3.5MM STANDARD FLEXTEND PLUS	4374442
60PFS35	PAED. TRACHEOSTOMY TUBE 3.5MM STANDARD FLEXTEND PLUS	4381019
60PFS40	PAED. TRACHEOSTOMY TUBE 4.0MM STANDARD FLEXTEND PLUS	4048022
60PFS40	PAED. TRACHEOSTOMY TUBE 4.0MM STANDARD FLEXTEND PLUS	4348476
60PFS40	PAED. TRACHEOSTOMY TUBE 4.0MM STANDARD FLEXTEND PLUS	4357235
60PFS45	PAED. TRACHEOSTOMY TUBE 4.5MM STANDARD FLEXTEND PLUS	3901292
60PFS45	PAED. TRACHEOSTOMY TUBE 4.5MM STANDARD FLEXTEND PLUS	3962597
60PFS45	PAED. TRACHEOSTOMY TUBE 4.5MM STANDARD FLEXTEND PLUS	3982681
60PFS45	PAED. TRACHEOSTOMY TUBE 4.5MM STANDARD FLEXTEND PLUS	3988104
60PFS45	PAED. TRACHEOSTOMY TUBE 4.5MM STANDARD FLEXTEND PLUS	3988105
60PFS45	PAED. TRACHEOSTOMY TUBE 4.5MM STANDARD FLEXTEND PLUS	3991169
60PFS45	PAED. TRACHEOSTOMY TUBE 4.5MM STANDARD FLEXTEND PLUS	4322676
60PFS45	PAED. TRACHEOSTOMY TUBE 4.5MM STANDARD FLEXTEND PLUS	4345015
60PFS45	PAED. TRACHEOSTOMY TUBE 4.5MM STANDARD FLEXTEND PLUS	4347208
60PFS45	PAED. TRACHEOSTOMY TUBE 4.5MM STANDARD FLEXTEND PLUS	4351241
60PFS45	PAED. TRACHEOSTOMY TUBE 4.5MM STANDARD FLEXTEND PLUS	4353095
60PFS45	PAED. TRACHEOSTOMY TUBE 4.5MM STANDARD FLEXTEND PLUS	4379941
60PFS45	PAED. TRACHEOSTOMY TUBE 4.5MM STANDARD FLEXTEND PLUS	4381022
60PFS45	PAED. TRACHEOSTOMY TUBE 4.5MM STANDARD FLEXTEND PLUS	4388178
60PFS45	PAED. TRACHEOSTOMY TUBE 4.5MM STANDARD FLEXTEND PLUS	4395789
60PFS50	PAED. TRACHEOSTOMY TUBE 5.0MM STANDARD FLEXTEND PLUS	3991176
60PFS50	PAED. TRACHEOSTOMY TUBE 5.0MM STANDARD FLEXTEND PLUS	3997828
60PFS50	PAED. TRACHEOSTOMY TUBE 5.0MM STANDARD FLEXTEND PLUS	4212782
60PFS50	PAED. TRACHEOSTOMY TUBE 5.0MM STANDARD FLEXTEND PLUS	4218709
60PFS50	PAED. TRACHEOSTOMY TUBE 5.0MM STANDARD FLEXTEND PLUS	4286805
60PFS50	PAED. TRACHEOSTOMY TUBE 5.0MM STANDARD FLEXTEND PLUS	4289485
60PFS50	PAED. TRACHEOSTOMY TUBE 5.0MM STANDARD FLEXTEND PLUS	4365778
60PFS50	PAED. TRACHEOSTOMY TUBE 5.0MM STANDARD FLEXTEND PLUS	4366203
60PFS50	PAED. TRACHEOSTOMY TUBE 5.0MM STANDARD FLEXTEND PLUS	4369601
60PFS50	PAED. TRACHEOSTOMY TUBE 5.0MM STANDARD FLEXTEND PLUS	4389904
60PFS50	PAED. TRACHEOSTOMY TUBE 5.0MM STANDARD FLEXTEND PLUS	4395790
60PFS55	PAED. TRACHEOSTOMY TUBE 5.5MM STANDARD FLEXTEND PLUS	3868208
60PFS55	PAED. TRACHEOSTOMY TUBE 5.5MM STANDARD FLEXTEND PLUS	3868209
60PFS55	PAED. TRACHEOSTOMY TUBE 5.5MM STANDARD FLEXTEND PLUS	3930034
60PFS55	PAED. TRACHEOSTOMY TUBE 5.5MM STANDARD FLEXTEND PLUS	3939625
60PFS55	PAED. TRACHEOSTOMY TUBE 5.5MM STANDARD FLEXTEND PLUS	3982685
60PFS55	PAED. TRACHEOSTOMY TUBE 5.5MM STANDARD FLEXTEND PLUS	3988111
60PFS55	PAED. TRACHEOSTOMY TUBE 5.5MM STANDARD FLEXTEND PLUS	3988112
60PFS55	PAED. TRACHEOSTOMY TUBE 5.5MM STANDARD FLEXTEND PLUS	3991179
60PFS55	PAED. TRACHEOSTOMY TUBE 5.5MM STANDARD FLEXTEND PLUS	3997818
60PFS55	PAED. TRACHEOSTOMY TUBE 5.5MM STANDARD FLEXTEND PLUS	3997829
60PFS55	PAED. TRACHEOSTOMY TUBE 5.5MM STANDARD FLEXTEND PLUS	4148016
60PFS55	PAED. TRACHEOSTOMY TUBE 5.5MM STANDARD FLEXTEND PLUS	4204820
60PFS55	PAED. TRACHEOSTOMY TUBE 5.5MM STANDARD FLEXTEND PLUS	4322677
60PFS55	PAED. TRACHEOSTOMY TUBE 5.5MM STANDARD FLEXTEND PLUS	4335021

60PFSS45	4.5MM FLEXTEND PEDIATRIC STANDARD STRAIGHT TRACH TUBE	4379948
60SN040	4.0 UNCUFFED NEONATAL TRACHEOSTOMY TUBE	3952369
65P025	PAED. TRACHEOSTOMY TUBE 2.5MM AIRE-CUF	3981971
65P030	PAED. TRACHEOSTOMY TUBE 3.0MM AIRE-CUF	3930042
65P035	PAED, TRACHEOSTOMY TUBE 3.5mm AIRE-CUF	4035680
65P040	PAED. TRACHEOSTOMY TUBE 4.0MM AIRE-CUF	3952347
65P040	PAED. TRACHEOSTOMY TUBE 4.0MM AIRE-CUF	3965907
65P040	PAED. TRACHEOSTOMY TUBE 4.0MM AIRE-CUF	4153617
65P040	PAED. TRACHEOSTOMY TUBE 4.0MM AIRE-CUF	4237344
65P040	PAED. TRACHEOSTOMY TUBE 4.0MM AIRE-CUF	4289465
65P045	PAED. TRACHEOSTOMY TUBE 4.5MM AIRE-CUF	3868312
65P045	PAED. TRACHEOSTOMY TUBE 4.5MM AIRE-CUF	3904682
65P045	PAED. TRACHEOSTOMY TUBE 4.5MM AIRE-CUF	3933472
65P050	PAED. TRACHEOSTOMY TUBE 5.0MM AIRE-CUF	3942493
65P050	PAED. TRACHEOSTOMY TUBE 5.0MM AIRE-CUF	4163824
65P050	PAED. TRACHEOSTOMY TUBE 5.0MM AIRE-CUF	4179335
65P050	PAED. TRACHEOSTOMY TUBE 5.0MM AIRE-CUF	4221325
65P050	PAED. TRACHEOSTOMY TUBE 5.0MM AIRE-CUF	4334867
65P055	PAED. TRACHEOSTOMY TUBE 5.5MM AIRE-CUF	4056231
67N040	TTS NEONATAL TRACHY TUBE 4.0MMTIGHT TO SHAFT CUFF	3901313
67P040	TTS PEDIATRIC TRACH TUBE	4086033
67P045	TTS PEDIATRIC TRACH TUBE	3946275
67P045	TTS PEDIATRIC TRACH TUBE	3946276
67P045	TTS PEDIATRIC TRACH TUBE	3955778
67P045	TTS PEDIATRIC TRACH TUBE	3959301
67P045	TTS PEDIATRIC TRACH TUBE	4063165
67P045	TTS PEDIATRIC TRACH TUBE	4131003
67P045	TTS PEDIATRIC TRACH TUBE	4159971
67P045	TTS PEDIATRIC TRACH TUBE	4273416
67P050	TTS PEDIATRIC TRACH TUBE	4005585
67P050	TTS PEDIATRIC TRACH TUBE	4018498
67P050	TTS PEDIATRIC TRACH TUBE	4048028
67P050	TTS PEDIATRIC TRACH TUBE	4060210
67P050	TTS PEDIATRIC TRACH TUBE	4077095
67P050	TTS PEDIATRIC TRACH TUBE	4128019
67P050	TTS PEDIATRIC TRACH TUBE	4142417
67P050	TTS PEDIATRIC TRACH TUBE	4237352
67P050	TTS PEDIATRIC TRACH TUBE	4344974
67P050	TTS PEDIATRIC TRACH TUBE	4352766
67P050	TTS PEDIATRIC TRACH TUBE	4353078
67P055	TTS PEDIATRIC TRACH TUBE	4063167
67P055	TTS PEDIATRIC TRACH TUBE	4085996
67P055	TTS PEDIATRIC TRACH TUBE	4327919
67P055	TTS PEDIATRIC TRACH TUBE	4380071

67PFP60	PEDIATRIC TRACHEOSTOMY TUBE 6.0mm TTS FLEXTEND PLUS V FLANGE	3828304
67PFP60	PEDIATRIC TRACHEOSTOMY TUBE 6.0mm TTS FLEXTEND PLUS V FLANGE	3962593
67PFP60	PEDIATRIC TRACHEOSTOMY TUBE 6.0mm TTS FLEXTEND PLUS V FLANGE	4035699
67PFP60	PEDIATRIC TRACHEOSTOMY TUBE 6.0mm TTS FLEXTEND PLUS V FLANGE	4060222
67PFP60	PEDIATRIC TRACHEOSTOMY TUBE 6.0mm TTS FLEXTEND PLUS V FLANGE	4086014