08 lutego 2023 r.

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Z poważaniem,

Lorna Darrock  
 Associate Director, Post Market Quality  
 EMEA Quality

BD Switzerland Sàrl  
Terre Bonne Park – A4  
Route de Crassier 17  
1262 Eysins – Switzerland  
Tél: +41 21 556 30 Fax:  
+41 21 556 30 99  
www.BD.com

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SRN producenta: IL-MF-000030321

Kod UDI-DI produktu: 0885403BIRZRZSODS

| Nr katalogowy (REF) | Opis produktu  |
|---------------------|--|
| 7290012271014       | BD BodyGuard™ MicroSet Microbore Tubing, Drip Chamber, 2 Y Ports, Anti Siphon Valve, Male Luer Connector                             |
| 7290012271021       | IV SET, LIGHT SENSITIVE DRUG, REMOVABLE AFF VALVE  |
| 7290012271045       | MicroSet Blood; Blood Set, Microbore Tubing, Drip Chamber, 2 Y Ports, Anti Siphon Valve, Female Luer Connector, Male Luer Connector  |
| 7290012271069       | BD BodyGuard™ MicroSet Blue Stripe Microbore Tubing, Inline 0.2 Um Filter, Anti Siphon Valve, Male Luer Connector                    |
| 100-160XE           | BD BodyGuard™ MicroSet Yellow Striped Microbore Tubing, Spike, Anti Siphon Valve, Male Luer Connector                                |
| 100-160XSMG90EK     | BD BodyGuard™ MicroSet Grey Microbe Tubing, 90° Spike, Anti-Siphon Valve, Male Luer Connector  |
| 100-163X2YNKS       | MICROSET, MRI, NON-DEHP, PRIMARY, VSA  |
| 100-163XE90SK       | BD BodyGuard™ MicroSet Yellow Striped Microbore Tubing, 90° Spike, Anti Siphon Valve, Male Luer Connector                            |
| 100-163XESK         | BD BodyGuard™ MicroSet Yellow Striped Microbore Tubing, Spike, Anti Siphon Valve, Male Luer Connector                                |
| 100-163XESVK        | BD BodyGuard™ MicroSet Yellow Striped Microbore Tubing, Spike, Anti Siphon Valve, Male Luer Connector                                |
| 100-163XN2YNKS      | BD BodyGuard™ MicroSet Microbore Tubing, Female Luer Connector, Needle Free Port, Y Ports, Anti Siphon Valve, Male Luer Connector    |
| 100-163XNKS         | BD BodyGuard™ MicroSet Syringe; Microbore Tubing, Syringe Adapter, Anti Siphon Valve, Male Luer Connector                            |
| 100-172SCHK         | SYR SET, MICROBORE, Y, FLL, PINCH CLAMP, NDEHP   |
| 100-184XSK          | BD BodyGuard™ MicroSet Blue Microbore Tubing, Syringe Adapter, Anti-Siphon Valve, Male Luer Connector                                |
| 100-184XSYK         | BD BodyGuard™ MicroSet Blue Microbore Tubing, Syringe Adapter, Y Port, Female Luer Connector, Anti-Siphon Valve, Male Luer Connector |
| 101-163XE90SK       | BD BodyGuard™ MicroSet Microbore Tubing, 90° Spike, Anti Siphon Valve, Male Luer Connector   |
| 120-000DBLSE        | MICROSET, NON-DEHP, BLOOD, SR, Y,180mic FILTER   |
| 120-000DBLSK        | MicroSet Blood; Blood Set, Microbore Tubing, Drip Chamber, Y Port, Anti Siphon Valve, Male Luer Connector                            |
| 120-000TF           | BD BodyGuard™ MicroSet Microbore Tubing, 2 Spikes, Inline 1.2 µm Filter, Anti-Siphon Valve, Male Luer Connector                      |
| 120-000XYLS         | MICROSET, NON-DEHP, LIGHT RESISTANT, Y PORT  |
| 120-003XS2YV        | BD BodyGuard™ MicroSet Tubing, Vented Spike, Y Port, Anti-Siphon Valve, Male Luer Connector  |
| 120-112XPEFKY       | BD BodyGuard™ MicroSet Low Sorbing Microbore Tubing, Spike, Inline 1.2 µm Filter, Y Port, Anti Siphon Valve, Male Luer Connector     |

BD Switzerland Sàrl  
 Terre Bonne Park – A4  
 Route de Crassier 17  
 1262 Eysins – Switzerland  
 Tél: +41 21 556 30 Fax:  
 +41 21 556 30 99  
 www.BD.com

| Catalogue (REF) No. | Product Description  |
|---------------------|--|
| 120-112XSFK         | BD BodyGuard™ MicroSet Microbore Tubing, Spike, Inline 1.2 µm Filter, Anti Siphon Valve, Male Luer Connector                       |
| 120-112XSFKV        | BD BodyGuard™ MicroSet Microbore Tubing, Spike, Manual Priming Valve, Inline 1.2 µm Filter, Anti Siphon Valve, Male Luer Connector |
| 120-112XSFGY        | MICROSET, GRAVITY, NEEDLES   |
| 120-124XSFK         | BD BodyGuard™ MicroSet Microbore Tubing, Spike, Y Port, Inline 1.2 µm Filter, Anti-Siphon Valve, Male Luer Connector               |
| 120-124XSK          | BD BodyGuard™ MicroSet Microbore Tubing, Spike, Needle- free Port, Back Check Valve, Anti-Siphon Valve, Male Luer Connector        |
| 120-160X90SK        | BD BodyGuard™ MicroSet Blue Striped Microbore Tubing, 90° Spike, Anti Siphon Valve, Male Luer Connector                            |
| 120-160XCSEK        | BD BodyGuard™ MicroSet Blue Striped Microbore Tubing, Spike, Y Port, Female Luer Connector, Anti Siphon Valve, Male Luer Connector |
| 120-160XCSK         | BD BodyGuard™ MicroSet Blue Striped Microbore Tubing, Spike, T Port, Female Luer Connector, Anti-Siphon Valve, Male Luer Connector |
| 120-160XPS          | IV SET, MICROSET, BLUE ST,80 IN, NDEHP, SR   |
| 120-160XSFD         | IV SET, MICROSET, NDEHP,0.2M FILTER, SR  |
| 120-160XSFK         | IV SET, MICROSET, NDEHP,0.2M FIL, SR, STRET  |
| 120-160XSFMK        | BD BodyGuard™ MicroSet Microbore Tubing, Spike, Inline 1.2 µm Filter, Anti Siphon Valve, Male Luer Connector                       |
| 120-160XYSE         | MICROSET, Y, SR SEGMENT, NON-DEHP, DRIP CHAMBER  |
| 120-160XYSK         | BD BodyGuard™ MicroSet Microbore Tubing, Drip Chamber, Y Port, Anti Siphon Valve, Male Luer Connector                              |
| 121-160XCSEK        | BD BodyGuard™ MicroSet Blue Striped Microbore Tubing, Spike, Y Port, Female Luer Connector, Anti Siphon Valve, Male Luer Connector |
| 130-163XE90SK       | BD BodyGuard™ MicroSet Green Microbore Tubing, 90° Spike, Anti Siphon, Male Luer Connector   |
| 410064-INTEGRA      | IP BODYSET,120-000XY NON-DEHP, NEEDLE FREE   |
| 410066-INTEGRA      | IP BODYSET,120-000XY, NON-DEHP   |
| 410067-INTEGRA      | IP BODYSET,120-000XY, NON-DEHP   |
| 410243-INTEGRA      | IP FOTO BODYSET, 120-000XYL, LIGHT RESIST  |
| 410247-INTEGRA      | IP FOTO BODYSET, 120-000XYL, LIGHT RESIST  |
| EPI00001            | BD BodyGuard™ MicroSet Yellow Striped Microbore Tubing, Spike, Anti Siphon Valve, Male Luer Connector                              |
| EPI00002            | MICROSET, EPIDURAL, NON-DEHP   |
| EPI00003            | BD BodyGuard™ MicroSet Yellow Stripped Microbore Tubing, Spike, Inline 0.2 µm filter, Anti Siphon Valve, Male Luer Connector       |
| EPI00011-NRF        | BD BodyGuard™ MicroSet NR Fit Yellow Striped Microbore Tubing, 90° Spike, Anti Siphon Valve, Male NRFit Connector                  |
| EPI00012-NRF        | BD BodyGuard™ MicroSet NR Fit Yellow Microbore Tubing, 90° Spike, Anti Siphon Valve, Male NRFit Connector                          |
| GEN00001            | BD BodyGuard™ MicroSet Microbore Tubing, Drip Chamber, Anti Siphon Valve, Male Luer Connector                                      |
| GEN00002            | BD BodyGuard™ MicroSet Microbore Tubing, Drip Chamber, Needle Free Port, Anti-Siphon Valve, Male Luer Connector                    |
| GEN00004            | BD BodyGuard™ MicroSet Light Resistant Microbore Tubing, Needle Free Port, Anti Siphon Valve, Male Luer Connector                  |
| GEN00014            | BD BodyGuard™ MicroSet Microbore Tubing, Spike, Anti-Siphon Valve, Male Luer Connector   |
| M100-172S           | SYR SET,1.6X3 TUBE, 1000MM LONG, VALVE, NDEHP  |
| M100-172SB          | BD BodyGuard™ MicroSet Syringe Microbore Tubing, Female Luer Connector, Anti Siphon Valve, Male Luer Connector                     |
| M100-172SL          | BD BodyGuard™ MicroSet Syringe Microbore Tubing, Female Luer Connector, Anti Siphon Valve, Male Luer Connector                     |

BD Switzerland Sàrl  
 Terre Bonne Park – A4  
 Route de Crassier 17  
 1262 Eysins – Switzerland  
 Tél: +41 21 556 30 Fax:  
 +41 21 556 30 99  
 www.BD.com

| Catalogue (REF) No. | Product Description   |
|---------------------|---|
| ONC00002            | BD BodyGuard™ MicroSet Microbore Tubing, 2 Drip Chambers, Y Port, Anti Siphon Valve, Male Luer Connector                                |
| ONC00004            | BD BodyGuard™ MicroSet Low Sorbing Microbore Tubing, Vented Spike, Y Port, Back Check Valve, Male Luer Connector                        |
| ONC00004UV          | BD BodyGuard™ MicroSet Sorbing Light Resistant Microbore Tubing, Vented Spike, Y Port, Anti Siphon Valve, Male Luer Connector           |
| ONC00005            | BD BodyGuard™ MicroSet Low Sorbing Microbore Tubing, Drip chamber, Y Port, Anti Siphon Valve, Male Luer Connector                       |
| ONC00006            | BD BodyGuard™ MicroSet Low Sorbing Microbore Tubing, Drip Chamber, Inline 0.2 µm Filter, Y Port, Anti Siphon Valve, Male Luer Connector |
| ONC00008            | BD BodyGuard™ MicroSet Low Sorbing Microbore Tubing, Anti Siphon Valve, 5 SmartSite Ports, Male Luer Connector                          |
| ONC00010            | BD BodyGuard™ MicroSet Low Sorbing Light Resistant Microbore Tubing, Anti Siphon Valve 5 SmartSite Ports, Male Luer Connector           |
| ONC00011            | BD BodyGuard™ MicroSet Low Sorbing Microbore Tubing, Anti Siphon Valve, 3 SmartSite Ports, Male Luer Connector                          |
| ONC00012            | BD BodyGuard™ MicroSet Low Sorbing Light Resistant Microbore Tubing, Anti Siphon Valve 3 SmartSite Ports, Male Luer Connector           |
| PCA00001            | BD BodyGuard™ MicroSet Blue Striped Microbore Tubing, Spike, Anti Siphon Valve, Male Luer Connector                                     |
| PCA00002            | BD BodyGuard™ MicroSet Low Sorbing Microbore Tubing, Spike, Anti Siphon Valve, Male Luer Connector                                      |
| PCA00003            | BD BodyGuard™ MicroSet Low Sorbing Microbore Tubing, Spike, Inline 0.2 µm Filter, Anti Siphon Valve, Male Luer Connector                |
| PCA00004            | BD BodyGuard™ MicroSet Blue Striped Microbore Tubing, Spike, Anti-Siphon Valve, Male Luer Connector                                     |
| PCA00005            | BD BodyGuard™ MicroSet Blue Striped Microbore Tubing, 150ml EVA Bag, Inline 0.2 µm Filter, Anti-Siphon Valve, Male Luer Connector       |
| PCA00009            | MICROSET, PCA, NON-DEHP, DRIP CHAMBER, 15m FILTER   |
| PCA00013            | MICROSET, PCA, NON DEHP, Y FLUSH PORT, 0.2M FILTER  |
| TPN00001            | BD BodyGuard™ MicroSet Microbore Tubing, Spike, Inline 1.2 µm Filter, Anti Siphon Valve, Male Luer Connector                            |
| TPN00003            | BD BodyGuard™ MicroSet Microbore Tubing, Spike, Inline 1.2 µm Filter, Anti Siphon Valve, Male Luer Connector                            |
| TPN00004            | BD BodyGuard™ MicroSet Light Resistant Microbore Tubing, Inline 1.2 µm Filter, Anti Siphon Valve, Male Luer Connector                   |
| TPN00005            | BD BodyGuard™ MicroSet Microbore Tubing, Vented Spike, Inline 1.2 µm Filter, Anti Siphon Valve, Male Luer Connector                     |