---

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## Closing Confirmation Sheet

DRI OCT Triton, DRI OCT Triton plus Software, Ver.1.1.4

|               |                       |                 |
|---------------|-----------------------|-----------------|
| Customer name | Installation date / / |                 |
|               | Agency/Supplier       | Business office |
| Address       |                       |                 |
| TEL ( ) -     | The body model:       | The body S/N:   |

● Version Up check procedure of a software

|                           | Procedure item   | Check point   | PASS                     | FALL                     |                           |             |                          |                          |        |       |  |                   |  |  |                      |  |   |                          |                          |                          |                          |
|---------------------------|--|---|--------------------------|--------------------------|---------------------------|-------------|--------------------------|--------------------------|--------|-------|--|-------------------|--|--|----------------------|--|---|--------------------------|--------------------------|--------------------------|--------------------------|
| 1                         | It be Ver1.1.3 in front of Version Up.<br>(Select "information" from the "SETUP" menu.)<br><table border="1" style="margin-left: 20px;"> <tr> <td>Basic</td> <td>Serial No</td> <td></td> </tr> <tr> <td></td> <td>Region Code</td> <td></td> </tr> <tr> <td></td> <td>System</td> <td>1.1.3</td> </tr> <tr> <td></td> <td>OCT Capture Count</td> <td></td> </tr> <tr> <td></td> <td>Fundus Capture Count</td> <td></td> </tr> </table>  | Basic   | Serial No                |                          |                           | Region Code |                          |                          | System | 1.1.3 |  | OCT Capture Count |  |  | Fundus Capture Count |  | It's confirmed that it's a repair target.   | <input type="checkbox"/> | <input type="checkbox"/> |                          |                          |
| Basic                     | Serial No  |   |                          |                          |                           |             |                          |                          |        |       |  |                   |  |  |                      |  |   |                          |                          |                          |                          |
|                           | Region Code  |   |                          |                          |                           |             |                          |                          |        |       |  |                   |  |  |                      |  |   |                          |                          |                          |                          |
|                           | System   | 1.1.3   |                          |                          |                           |             |                          |                          |        |       |  |                   |  |  |                      |  |   |                          |                          |                          |                          |
|                           | OCT Capture Count  |   |                          |                          |                           |             |                          |                          |        |       |  |                   |  |  |                      |  |   |                          |                          |                          |                          |
|                           | Fundus Capture Count   |   |                          |                          |                           |             |                          |                          |        |       |  |                   |  |  |                      |  |   |                          |                          |                          |                          |
| 2                         | Version Up, error confirmation   | An error message don't be indicated.  | <input type="checkbox"/> | <input type="checkbox"/> |                           |             |                          |                          |        |       |  |                   |  |  |                      |  |   |                          |                          |                          |                          |
| 3                         | Version behind Version Up be Ver.1.1.4.<br>(Select "information" from the "SETUP" menu.)<br><table border="1" style="margin-left: 20px;"> <tr> <td>Basic</td> <td>Serial No</td> <td></td> </tr> <tr> <td></td> <td>Region Code</td> <td></td> </tr> <tr> <td></td> <td>System</td> <td>1.1.4</td> </tr> <tr> <td></td> <td>OCT Capture Count</td> <td></td> </tr> <tr> <td></td> <td>Fundus Capture Count</td> <td></td> </tr> </table> | Basic   | Serial No                |                          |                           | Region Code |                          |                          | System | 1.1.4 |  | OCT Capture Count |  |  | Fundus Capture Count |  | Make sure that the version information is as follows.<br><table border="1" style="margin-left: 20px;"> <tr> <td>System</td> <td>1.1.4</td> </tr> </table> | System                   | 1.1.4                    | <input type="checkbox"/> | <input type="checkbox"/> |
| Basic                     | Serial No  |   |                          |                          |                           |             |                          |                          |        |       |  |                   |  |  |                      |  |   |                          |                          |                          |                          |
|                           | Region Code  |   |                          |                          |                           |             |                          |                          |        |       |  |                   |  |  |                      |  |   |                          |                          |                          |                          |
|                           | System   | 1.1.4   |                          |                          |                           |             |                          |                          |        |       |  |                   |  |  |                      |  |   |                          |                          |                          |                          |
|                           | OCT Capture Count  |   |                          |                          |                           |             |                          |                          |        |       |  |                   |  |  |                      |  |   |                          |                          |                          |                          |
|                           | Fundus Capture Count   |   |                          |                          |                           |             |                          |                          |        |       |  |                   |  |  |                      |  |   |                          |                          |                          |                          |
| System                    | 1.1.4  |   |                          |                          |                           |             |                          |                          |        |       |  |                   |  |  |                      |  |   |                          |                          |                          |                          |
| 4                         | Checking existing data   | Check existing data by the tool software "PowerSaveLRChecker". If wrong data exist make necessary correction.<br><table border="1" style="margin-left: 20px;"> <tr> <td>Nuber of checked images</td> <td></td> </tr> <tr> <td>Hours taken for the check</td> <td></td> </tr> </table> | Nuber of checked images  |                          | Hours taken for the check |             | <input type="checkbox"/> | <input type="checkbox"/> |        |       |  |                   |  |  |                      |  |   |                          |                          |                          |                          |
| Nuber of checked images   |  |   |                          |                          |                           |             |                          |                          |        |       |  |                   |  |  |                      |  |   |                          |                          |                          |                          |
| Hours taken for the check |  |   |                          |                          |                           |             |                          |                          |        |       |  |                   |  |  |                      |  |   |                          |                          |                          |                          |

|                |
|----------------|
| <b>Comment</b> |
|----------------|

| Name                        | Signature | Initials | Date |
|-----------------------------|-----------|----------|------|
| <b>TESTING PERFORMED BY</b> |           |          |      |
|                             |           |          |      |
| <b>CUSTOMER</b>             |           |          |      |
|                             |           |          |      |