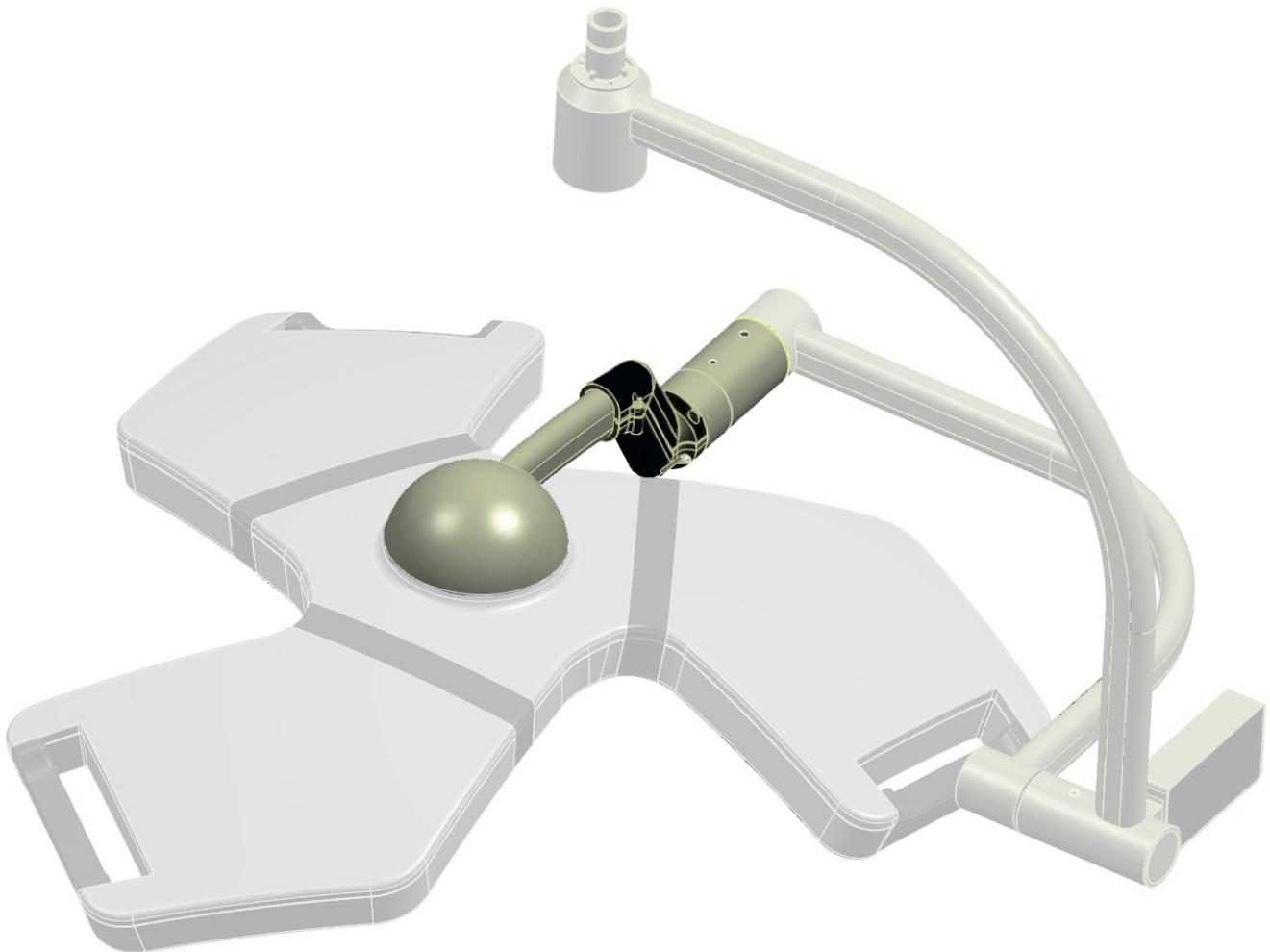
TRILUX
ENGINEERING HEALTH.

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en **Mounting instructions for**

Aurinio® Leuchtensicherung **Aurinio® light protection kit**

TK 10153091 Stand 05/17

Copyright © 2017 TRILUX Medical GmbH & Co. KG

TRILUX Medical GmbH & Co. KG

Huettenstrasse 21

D-59759 Arnsberg

Tel. +49 (0) 29 32 / 9214 - 0

Fax +49 (0) 29 32 / 9214 - 501

info@trilux-medical.com

www.trilux-medical.com

DE

Inhalt

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Übersicht

Hinweise

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Montage

EN

Contents

Statement of purpose for accessories

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Statement of purpose for accessories

These accessories are tested and authorised exclusively for utilisation on and/or with **Aurinio® operating theatre luminaires** made by **TRILUX Medical**.

It serves as additional protection for the luminaire connection.

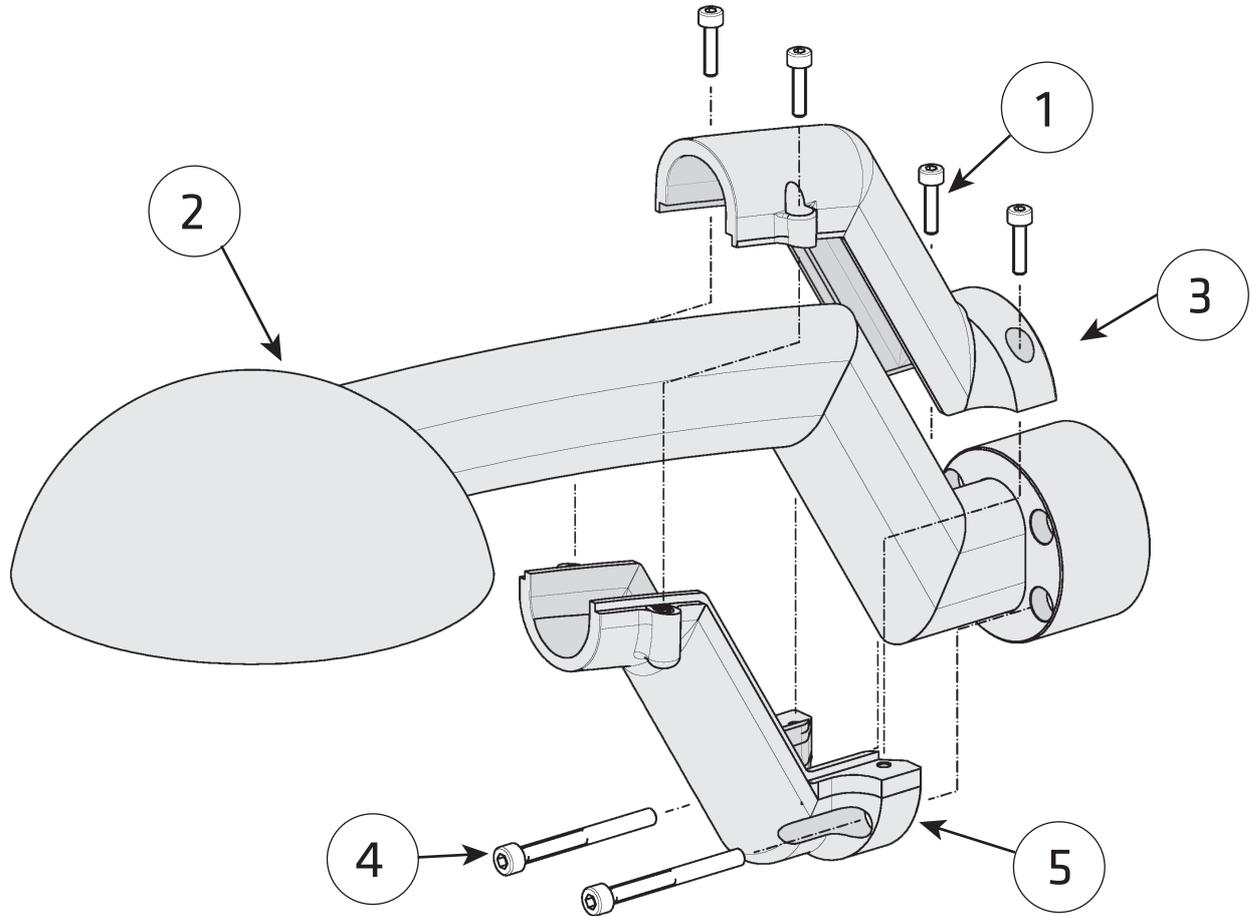
Quotation from the Operator's Ordinance:

§2(1): Medicine products may only be installed, operated and used in accordance with its intended purpose and in accordance with the provisions of this Ordinance, the generally recognized rules of technology, and the occupational safety and accident prevention regulations.

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EN

Notes

Please examine the device and the accessory parts for transport damage and deficiencies immediately after receipt.

Claims for damage will be honoured only if the seller (**TRILUX Medical GmbH & Co. KG**) or the transport company is informed at once.

A damage report must then be compiled at once.

The damage report must be sent to a representative of **TRILUX Medical GmbH & Co. KG** in your vicinity or directly to **TRILUX Medical GmbH & Co. KG**.

The original packaging should be used wherever possible when returning a luminaire or parts of a luminaire. If this packaging is no longer available for use, then an adequate packaging substitute must be found in order to avoid damage during transport. The following accompanying documentation must be enclosed with the shipment:

- Name and address of the user,
- Parts number and serial number (see type plate),
- Description of the deficiency

Please examine the device and the accessory parts for transport damage and deficiencies immediately after receipt. Claims for damage will be honoured only if the seller (**TRILUX Medical GmbH & Co. KG**) or the transport company is informed at once.

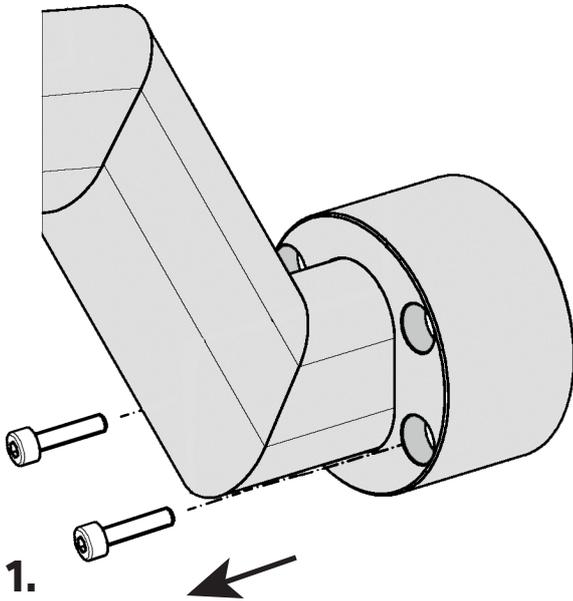
Keep these mounting instructions for any future maintenance work or dismantling, or make these mounting instructions available to the appropriate personnel.

Safety notes

- The installation is only provided for trained, expert technical personnel.
- The safety and reliability of the **Aurinio® OP-system** is only ensured if original **TRILUX Medical** parts are used.
- The **Aurinio® OP-system** is not intended for operation in areas at risk of explosion.

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EN
Mounting

1.

Remove the two lower allen screws at the cardanic connection of the luminaire connection.

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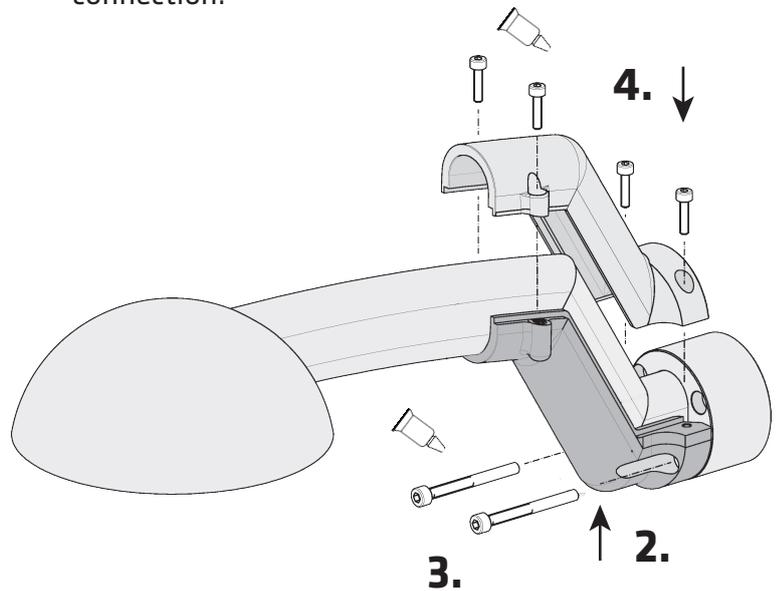
2.

- 2.**
Attach the lower support shell to the luminaire connection.

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Note:
Secure all screws with the supplied screw adhesive **Weicon Lock AN 302-43**. Fully wet the threads!

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- 3.**
Secure with allen screws M5 x 50. Tighten the screws with 4.2 Nm.

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- 4.**
Fit upper shell and secure with allen screws M4 x 16 (tightening torque 2.2 Nm).