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Cylindrical shape. Smooth condoms made of extremely thin, resistant, natural rubber latex. Lubricated and equipped with reservoir. Store in cool, dry place away from direct sunlight. Read and follow the illustrated instructions carefully.

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Cylindrical shape. Smooth condoms made of extremely thin, resistant, natural rubber latex. Lubricated and equipped with reservoir.

Store in cool, dry place away from direct sunlight. Read and follow the illustrated instructions carefully.

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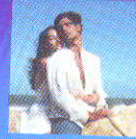
واقى طبيعي
3
PRESERVATIFS CLASSIQUES
CLASSIC CONDOMS

CONTROLE PUR®
3
واقى طبيعي
PRESERVATIFS CLASSIQUES
CLASSIC CONDOMS

IN ORDER TO OBTAIN THE MAXIMUM PROTECTION, IT IS NECESSARY TO FOLLOW THE INSTRUCTION FOR USE

- Do not use the condom if its material is brittle, sticky or obviously damaged.
- Open the wrapper without using sharp objects after taking the condom far from the cut line and remove it paying attention not to damage it: a condom with nicks, frings, etc (Fig. 1).
- Put the condom on when the penis is in erection, before any contact between the penis and the partner body and not only before ejaculation.
- While placing the condom, squeeze the teardrop to expel the air of the reservoir, keeping an empty space between the penis and the end of the reservoir for the sperm (Fig. 2) and unroll the condom carefully covering the penis totally (Fig. 3).
- After ejaculation, slowly withdraw the penis while it is still erect holding the condom firmly in place at the base, to ensure that the condom does not slip off (Fig. 4).
- Dispose the used condom safely into the rubbish, not into the WC.
- If you wish more lubrication, use water-based lubricants (ex. Glycerine). Avoid the use of oil-based lubricants such as vaseline.

- paraffin, baby oil, body cream or lotion, massage oil, butter, margarine, etc, because they could damage the integrity of the condom.
- Consult a doctor or a pharmacist about the compatibility of the condom with topic products that are applied on the penis or into the vagina.
- Use a new condom for each sexual intercourse or if you change of partner during the same intercourse (it is a disposable product).
- Use the condom immediately after having opened the wrapper.
- In case of leakage, the sperm can enter into contact with the partner, therefore, if the condom is used to prevent pregnancy, it is advisable to consult a doctor or a Health Centre within the following 72 hours.
- Even if no contraceptive can ensure total safety, the condom, used correctly could prevent Sexually Transmitted Diseases (STD) such as syphilis, gonorrhoea, chlamydia infections, herpes genitalis, hepatitis B or AIDS.
- If you have injuries in your hands, avoid the direct contact with the condom, in order to reduce the possibilities of infection.
- Store the condom in a cool and dry place away from direct sunlight.
- This product should not be used by person allergic to latex.
- If a condom is used for anal intercourse the risk of breakage increases. For these cases use the Foris model.

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