
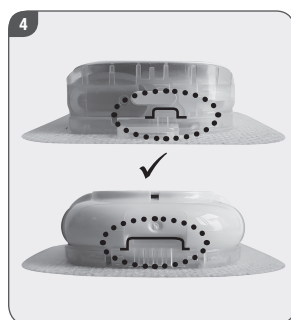
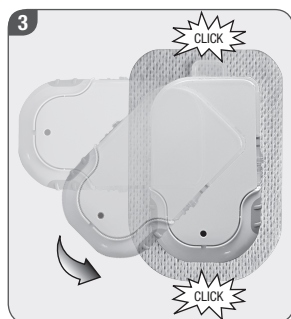
# ACCU-CHEK® Solo Micropump system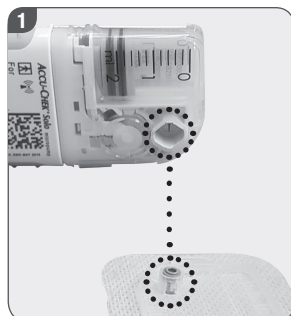

## EN Corrections to the User's Manual

### Accu-Chek Solo micropump system

#### ⚠ WARNING

##### Risk of hyperglycaemia (high blood glucose level)

The procedure for attaching the micropump that is described in chapter 4.3.6 of the User's Manual and that is illustrated on the diabetes manager, for example, in the *Attaching the micropump* help video, may lead to insufficient delivery of insulin.

Do not follow the procedure for attaching the micropump from the User's Manual or the diabetes manager. Instead, use only the following procedure.

#### Attaching the Micropump

- 1 Hold the micropump approximately at a 90 degree angle to the pump holder. Make sure the reservoir needle is above the grey cannula head of the pump holder.
- 2 Place the reservoir needle of the micropump on the grey cannula head of the pump holder.
- 3 Turn the micropump carefully into place. You may apply light pressure on the pump shield to make sure the reservoir needle stays inside the cannula head of the pump holder. After a quarter turn, the micropump clicks into the front and back hooks on the pump holder.
- 4 Examine the front and back hooks of the pump holder to make sure the micropump properly clicked into place.

#### ⚠ WARNING

##### Risk of hyperglycaemia (high blood glucose level)

Handling errors or defective components may lead to insufficient delivery of insulin.

Check your blood glucose level at least once within 1 to 3 hours after replacing a component of the micropump system.

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© 2019 Roche Diabetes Care
ACCU-CHEK and ACCU-CHEK SOLO are trademarks of Roche.